

Glucose


(GOD-POD Method)

Intended Use

Glucose is a reagent kit used for the determination of true glucose in serum or plasma, based on enzymatic method; using glucose oxidase and peroxidase enzymes.

Principle

Glucose oxidase (GOD) converts glucose into gluconic acid and Hydrogen peroxide (H_2O_2). Hydrogen peroxide, in presence of peroxidase (POD), oxidatively couples with 4-aminoantipyrine and phenol to produce red quinoneimine dye. This dye has absorbance maximum at 505 nm (500 - 550 nm). The intensity of the colored complex is directly proportional to the concentration of glucose in specimen.

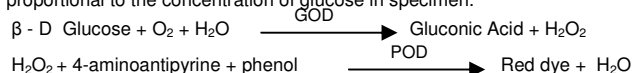

Reagents provided

1. Glucose enzyme reagent - Ready to use.
2. Standard - Glucose (100 mg/dl).

Reagent storage and stability

The kit should be stored at $2^\circ - 8^\circ\text{C}$ and is stable till the expiry date indicated on the label.

The reagent and standard are ready-to-use and are stable till expiry, when stored at $2^\circ - 8^\circ\text{C}$. DO NOT FREEZE THE REAGENT. The reagent should be stored only in the amber bottle provided to protect it from direct light. This is critical because the reagent is light sensitive (auto oxidation of chromogen system by light and air). Before use swirl the reagents gently. DO NOT SHAKE VIGOROUSLY. Over the period of storage, the reagent may develop a light pink colour. This is expected and does not affect the reagent performance. Discard the reagent if the absorbance of the same exceeds 0.300 against distilled water blank at 505 nm. Contamination of the reagent should be strictly avoided. If the reagent develops turbidity discard the reagent.

Specimen collection and preservation

Blood should be collected in a clean dry container. Serum or plasma should be separated from the cells at the earliest possible (within 30 minutes), as the rate of glycolysis is approximately 7 mg% per hour at room temperature ($25^\circ - 30^\circ\text{C}$).

For plasma separation following anticoagulants may be used.

- EDTA 2 mg/ml of blood
- CITRATE 6 mg/ml of blood
- HEPARINE 200 IU/ml of blood
- OXALATE 3 mg/ml of blood
- SODIUM FLUORIDE 10 mg/ml of blood

Sodium fluoride is preferred anticoagulant due to its antiglycolytic activity. Higher concentration of sodium fluoride i.e. more than 10 mg/ml of blood should be avoided as it may inhibit the colour development. Glucose is stable for 24 hours in neatly separated plasma and serum at $2^\circ - 8^\circ\text{C}$. If the estimation is not possible within 24 hours then the specimen should be preserved at -20°C and should be used within 30 days.

Assay guidelines for Analyzers

Reaction Type	End Point with Standard
Reaction slope	Increasing
Incubation time	7 min. at 37°C / 15 min. at RT ($25^\circ\text{-}30^\circ\text{C}$).
Wave length	505 nm (490 - 530 nm)
Blank	Reagent Blank
Blank absorbance limit	< 0. 300 Abs. against distilled water blank.
Sample Volume	10 μl (0.01 ml)
Reagent Volume	1000 μl (1.0 ml)
Glucose Standard Concentration	100 mg/dl
Factor Calculation	100 mg/dl $\div$ Absorbance of Standard.
Low normal	70 mg/dl
High normal	140 mg/dl
Linearity	Up to 500 mg/dl

Assay guidelines for Manual Procedure

Bring the enzyme reagent and standard to room temperature before performing the assay.

Reagents	Blank	Standard	Sample
Reagent	1000 µl (1.0 ml)	1000 µl (1.0 ml)	1000 µl (1.0 ml)
Standard	-	10 µl (0.01 ml)	-
Sample	-	-	10 µl (0.01 ml)

1. Mix thoroughly and incubate at 37°C for 7 minutes or 15 minutes at room temperature (25° - 30°C).
2. Read the absorbance against reagent blank at 505 nm (490 - 530nm).
3. The final colour is stable for 2 hours if not exposed to direct light.

Calculation

Glucose con.in sample (mg/dl) = $\frac{\text{Sample OD}}{\text{Std. OD.}} \times \text{Con. of Std.}$

Normal Range

Guidance value for Fasting serum / Plasma : 70 - 110 mg/dl

Guidance value for Post Prandial / Random : Up to 140 mg/dl

Note: Expected range varies from population to population and each laboratory should establish its own normal range.

Limitation

Reaction is linear up to 500 mg/dl. For higher values, dilute the sample with normal saline and perform the assay. Multiply the final result by dilution factor to get the real value.

Discard the reagent if the absorbance exceeds 0.300 against distilled water as blank at 505 nm.

Quality Control

To ensure adequate quality control measures, it is recommended that each batch should include a normal and an abnormal commercial reference control serum. It should be realized that the use of quality control material checks both instrument and reagent functions together. Factors which might affect the performance of this test include proper instrument function, temperature control, cleanliness of glassware and accuracy of pipetting.

References

1. Trinder P. Clin. Biochem. 6, 24 (1969).
2. Bergmayer H. V., Methods of enzymatic Analysis", A. P., N. Y. (1974). Page 1196.
3. Young, D.S., Pestaner, L.C., Gibberman, B., Clin. Chem. 21, 1D (1975).

Symbols

IVD

In Vitro Diagnostics.  Caution.



Keep away from sun light.



Date of Manufacture.

LOT

Batch No.



Read Instructions.



Fragile.



Product Expiry Date.

CONT

Content.



Storage Temperature.



Keep Dry.



Manufactured By.

REF

Catalogue No.



Kee Diagnostics
BIOCHEMICAL ANALYTICALS



KEE DIAGNOSTICS PVT LTD

CIN: U24231DL2004PTC128343 .

A subsidiary of KEE PHARMA LTD

A-8 . 3RD FLOOR. NARAINA INDUSTRIAL AREA, PHASE 2.

NEW DELHI-110028.

W. www.keediagnosics.com.

E. info@keediagnosics.com, T. 011 43136000.



ISO9001: 2015 & ISO 13485: 2016 Certified Company